



For your safety and
patient comfort

VITREX® STERILANCE® Lite II SAFETY LANCET

- ◆ USER SAFETY: Needle is fully shielded before and after use
- ◆ USER CONVENIENCE: Pre-loaded, button operated design
- ◆ PATIENT SAFETY: Single-use only, sterilised by gamma irradiation, cannot be re-used
- ◆ PATIENT COMFORT: High quality, ultra sharp tri-bevel needle, high speed penetration
- ◆ EFFICACY: Variety of point configurations and penetration depths to suit most capillary blood requirements
- ◆ APPLICATION: Blood glucose and other in vitro diagnostic tests

COLOUR CODE	CONFIGURATION	PENETRATION DEPTH	ARTICLE NO.	APPLICATION
Purple	28G needle	1.8 mm	374113	Diabetes clinics/hospital wards for blood glucose monitoring
Yellow	26G needle	1.8 mm	375113	Diabetes clinics/hospital wards for blood glucose monitoring
Blue	26G needle	2.4 mm	376113	As above but for thicker adult skin.
Orange	21G needle	1.8 mm	377113	Haemoglobin (blood banks), cholesterol, blood typing, coagulation, blood gas and other diagnostic tests. For paediatrics and most normal adult skin
Pink	21G needle	2.4 mm	378113	As above but for thicker adult skin.

Shelf life 5 years




- ▶ Twist off the protective cap

- ▶ **WARNING:** Do not use if protective cap has previously been removed



- ▶ Select fingertip insertion site slightly off centre to minimise any discomfort and maximise blood flow
- ▶ Hold the lancet firmly against site and press the release button to activate the device

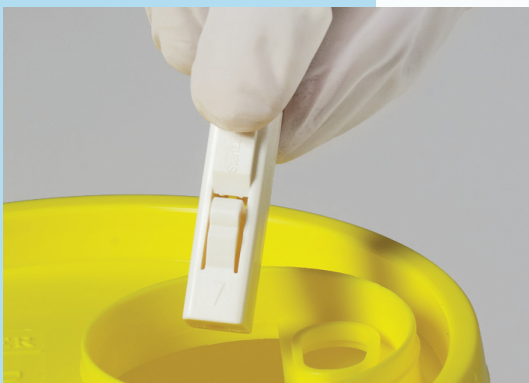

- ▶ Dispose of safety lancet in a suitable safety container



- ▶ Gently massage the finger in the direction of the puncture site to obtain the required volume of blood

PACKAGING

Primary pack: 100 pcs
 Shipper: 10 x 100 primaries
 Pieces per shipper: 1,000 pcs
 Pallet: 84,000 pcs